

Description:

ScarPro gel is a medical grade, fast drying, non-sensitizing silicone gel with vitamin E. **ScarPro** gel helps soften, flatten, smooth and fade scars.

- Effective on all types of scars, both old and new.
- Formulated for use on all skin types.
- Gentle and non-sensitizing, **ScarPro** gel is safe for use on children
- Relieves discomfort and itching associated with some scars.

ScarPro gel dries to form a thin, flexible, waterproof layer of silicone; creating a protective barrier with ideal environment to manage and reduce scarring.

ScarPro gel works through a combination of occlusion (seals the scar from exposure to air) and hydration.

ScarPro gel is ideal for use on the face, joints, knees, elbows and other areas requiring high flexibility.

Indication:

ScarPro gel is formulated to be used in the management of all types of scars, including hypertrophic and keloid scars, scars created by surgical procedures, scars from trauma/accidents and burn scars.

Contraindications:

ScarPro gel is contraindicated in patients with a known sensitivity to silicone.

DO NOT USE ON OPEN WOUNDS OR BROKEN SKIN.**General Directions for use:**

- Wash and dry the area being treated every time you apply **ScarPro** gel.
- Apply a thin layer of **ScarPro** gel to the area being treated and gently massage.
- Apply **ScarPro** gel at least twice a day. Reapplication is recommended after bathing, swimming or exercising.
- Once dry, **ScarPro** gel can be covered by clothing, cosmetics, sunblock and bandages.
- Store **ScarPro** gel in cool, dry place.

Recommended Treatment Period: 8 to 12 weeks.

Older scars may take longer.

Storage & Precautions:

Store below 25°C, For external use only. If a rash or itching occurs discontinue use and consult your doctor.

Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. Keep out of reach of children.

Presentation:

ScarPro gel is supplied in 6 gm and 15 gm tube.

Manufacturer & Marketing Authorisation Holder:

Middle East Pharmaceutical Industries Co Ltd (Avalon Pharma)

P.O.Box 4180 Riyadh 11491

2nd Industrial City, Riyadh, Kingdom of Saudi Arabia

Tel: +966 (11) 2653948 -2653427

Fax: +966 (11) 2654723

This leaflet was approved in [01/2023], Version No. 02